

Ori Sterilization

A clean tip mirror is critical for scanning success.



- Use sterilization wipe to disinfect the tip and body of the scanner.
- Do not touch the mirror with the sterilization wipe, as this could potentially damage the mirror and render the tip inoperable.
- To clean the tip's mirror, use a cotton-tipped applicator dipped in 95%+ isopropyl alcohol.
- Dry the mirror with a dry cotton tip applicator wrapped in a dry 2 × 2 gauze
- Fold a dry 2×2 gauze and place it in direct contact with the tip mirror. The gauze should be covering the entire mirror face.
- Place the scanner tip in a sterilization bag and autoclave it. (134°C for no more than 4 minutes)
- To sterilize the Ori Laptop, use 2×2 gauze with a small amount of isopropyl alcohol and gently wipe the keyboard.
- Do not put the scanner tips in the ultrasonic machine. Do not place the scanner tips in cold sterile. Doing so will damage the mirror making the tips inoperable.
- Inspect the scanner tips before each use, and discard if you notice any of the following:
 - *Cracks or large chips on the mirror*
 - *Tip becomes brittle*
 - *No longer fits firmly on the scanner*
 - *Has a loose fit*

