


# OriScan Path


## Quadrant

1. Start on the furthest molar and scan the occlusal surface.
2. Once you reach the midline rotate to the lingual and scan distally.
3. Slow roll over the furthest molar and wrap to the buccal and scan to the midline
4. Scan over any holes or voids.

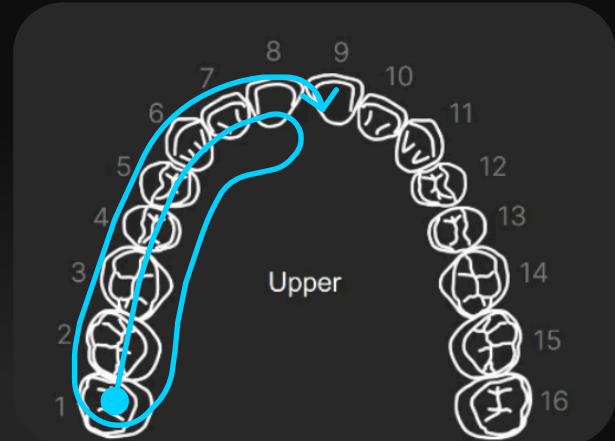

## Full Arch

1. Start directly on the furthest molar and complete a full arch occlusal scan.
2. Rotate lingually and complete full arch scan.
3. Rotate to the buccal and complete a full arch buccal scan.
4. Scan over any holes or voids.

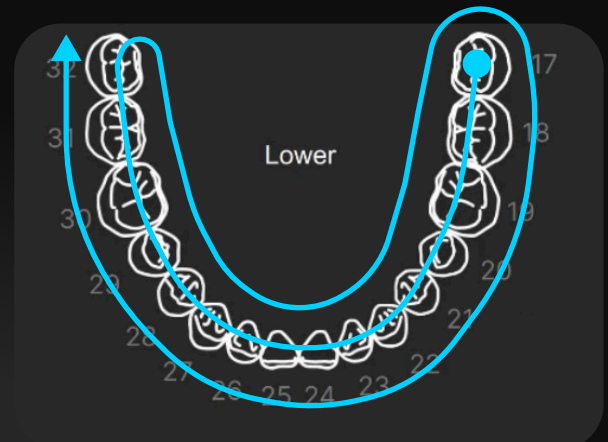

## Help & Support

- Submitting a support ticket at [oriscanning.com/support](https://oriscanning.com/support)
- Emailing your question to [support@oriscanning.com](mailto:support@oriscanning.com)
- Calling our support number at [\(435\) 500-2986](tel:(435)500-2986)